



**TT**

***Topmost Translucency  
Best Choice for Esthetic Restoration.***

*Topmost translucency: 55% transmittance, specially designed for  
Esthetic Restoration*

## Marvelously Performed on Zirconia Blocks

Through penetration throughout crown to ensure predictable colors

Highly match the natural colors of teeth

Shorter drying duration

Easy operation

## Application

Single unit(coping/full contour of anterior and posterior)

Veneer,Inlay/Onlay , 3 units anterior bridge

## Advantages

Bending strength: 750 Mpa

Excellent resistance to aging

Defect-free structure

Absolute biocompatibility

Precision sintering shrinkage

## TT Sintering Curve

| Sintering step | Temperature range(°C)   | Time(h)  |
|----------------|-------------------------|----------|
| paragraph 1    | 20-900                  | 1.5      |
| paragraph 2    | 900-900                 | 0.5      |
| paragraph 3    | 900-1450                | 3.0      |
| paragraph 4    | 1450-1450               | 2.0      |
| paragraph 5    | 1450-800                | 1.0      |
| paragraph 6    | 800-Natural cooling 100 | Finished |

